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UNITED STATES

Bayer Medical Care Inc.
1 Bayer Drive
Indianola, PA 15051-0780
U.S.A.
Phone: +1 (412) 767-2400
+1 (800) 633-7231
Fax: +1 (412) 767-4120

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©2010-2014, 2017 Bayer. All Rights Reserved. Reproduction of this manual is strictly prohibited without express written consent of Bayer Medical Care Inc.

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Bayer, the Bayer Cross, FluiDots, MEDRAD, Stellant and MEDRAD Stellant may be registered trademarks of Bayer in the US and other countries.

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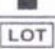
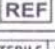
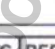
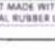
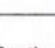
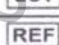
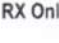
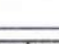
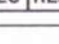
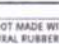
ENGLISH

Introduction: Read the information contained in this section. Understanding the information will assist you in operating the device in a safe manner.
Important Safety Notice: This device is intended to be used by individuals with adequate training and experience in diagnostic imaging studies.
Intended Use: The contents of this package are intended to be used in the delivery of contrast media or saline.
Contraindications: These devices are not intended for multiple patient use, drug infusion, chemotherapy, or any other use for which the device is not indicated.
Restricted Sale: Rx Only

Warnings

Air embolization can cause death or serious injury to the patient. Do not connect a patient to the injector until all trapped air has been cleared from the syringe and fluid path. Carefully read the instructions for loading and the use of FluidDots indicators (where applicable) to reduce the chance of air embolism.

Reuse of this product may result in biological contamination, product degradation and/or product performance issues. Properly discard disposable items after use, or if there is any possibility that contamination may have occurred.

Syringe sterility will be compromised, and patient infection may result, if the plunger is removed from the syringe. Do not remove the plunger to fill the syringe.

Bacterial contamination can occur if syringes are used to store contrast media. Use loaded syringes immediately. Discard unused loaded syringes.

For devices labeled for single use, please note: This product is intended for single use only. Do not resterilize, reprocess or reuse. The disposable devices have been designed an validated for single use only. Re-use of the single use disposable devices pose risks of device failure and risks to the patient. Potential device failure includes significant component deterioration with extended use, component malfunction, and system failure. Potential risks to the patient include injury due to device malfunction or infection as the device has not been validated to be cleaned or re-sterilized.

Do not use if sterile package is opened or damaged. Patient or operator injury may result if package is opened or damaged, or if damaged components are used. Visually inspect contents and package before each use.

Patient or operator injury may result from contrast media leaks or tubing ruptures. Ensure that the fluid path is open; do not exceed pressures identified on the front of the package. Use of greater pressures or occlusions in the fluid path may result in leaks or ruptures.

Patient injury could result if syringe is not properly engaged. Do not load or inject unless the syringe is properly engaged.

Use care in handling and inserting spike into contrast bottle. The spike is sharp and may cause personal injury.

Contamination may occur if the end of the spike is touched.

Do not use the transfer set on more than one container of media.

Do not use the transfer set for longer than six hours, once it has been exposed to media.

Patient or operator injury may result from damage to the transfer set valve. Do not use with injection flow rates higher than 4 mL/sec. Pressure generated by faster flow rates may cause damage to the valve resulting in leaks or ruptures.

Cautions

Component damage or leaks may occur if not installed properly. Ensure all connections are secure; do not overtighten. This will help minimize leaks, disconnection, and component damage.

Refer to the Injector Operation Manual for further instructions.

Note: To minimize bubbles when loading a syringe through a transfer set, do not exceed a reverse load rate of 7 mL/sec.

Installing a Syringe

Insert the syringe until it snaps into place. The piston automatically engages the plunger, and advances it to the front of the syringe (most models). To reduce the size and quantity of air bubbles drawn into the syringe during loading, a Fluid Dispensing Device from Bayer,

is recommended. Operator vigilance and care, coupled with a set procedure, is essential to minimizing the possibility of an air embolism. Point the injector head up during loading. Point the injector head down during an injection.

To help avoid an air injection, syringes from Bayer are equipped with FluidDots indicators. FluidDots indicators should be observed as part of an arming procedure. When the FluidDots are viewed through an empty syringe, the dots appear as small narrow ellipses. When viewed through a full syringe, the dots become larger, almost round. See below:

To minimize air embolization risks, ensure that one operator is designated the responsibility of filling the syringe(s). Do not change operators during the procedure. If an operator change must occur, ensure that the new operator verifies that the fluid path is purged of air.

Loading and Priming a Syringe Using a Spike or a Quick Fill Tube (QFT)

The syringe can be loaded manually or automatically on the MEDRAD Stellant injector. See the MEDRAD Stellant Operation Manual for more detailed instructions.

MEDRAD Stellant Manual Loading:

- 1. Install a new syringe.
2. Install the spike or QFT onto the end of the syringe. 1/4 turn to 1/2 turn maximum. Do not install with excessive force.
3. Insert the spike or QFT into the fluid source and fill the syringe with fluid, using the load buttons.
4. Expel air.
5. Remove spike or QFT. Connect the disposable tubing set.
6. Follow the instructions in the section "Connector Tube Installation."

MEDRAD Stellant Integral Autoload with a spike or QFT:

- 1. Set and lock protocol.
2. Install a new syringe.
3. Install the spike or QFT onto the end of the syringe. 1/4 turn to 1/2 turn maximum. Do not install with excessive force.
4. Insert the spike or QFT into the fluid source and press Autoload.
5. Press Fill.
6. Expel air.
7. Remove spike or QFT. Connect the disposable tubing set.
8. Follow the instructions in the section "Connector Tube Installation."
9. Press Start/Hold.

NOTE: Patency check through aspiration using the injector manual knob is not possible when using a check valve on the end of the connector tube. If aspiration is important, remove the check valve from the connector tube and directly connect the connector tube to the catheter.

Removing a Syringe(s)

- 1. Disconnect the disposable tubing set from the vascular entry device. The disposable tubing set does not need to be disconnected from the syringe.
2. Rotate the syringe approximately 1/4 turn counter-clockwise and gently pull the syringe out of the injector head, discarding the syringe with disposable tubing set.
NOTE: Once the syringe is removed from the injector, the piston will automatically retract (most models).
NOTE: In order to remove the syringe, the last piston motion must be in the forward direction, which is typical. If you cannot remove the syringe, rotate the injector head manual knob approximately one revolution in the forward direction, then repeat Step 2.

Connector Tube Installation

- 1. Remove the connector tube from the package, removing the dust covers on the luer fittings.
2. Ensure all air is purged from the syringe.
3. Attach the connector tube to the syringe, 1/4 turn to 1/2 turn maximum. Do not install with excessive force.
4. Ensure that the connector luer fitting is secured to the tip of the syringe and verify that the tubing is not kinked or obstructed.
If you are using a connector tube with T-connector, attach the straight portion of the T-connector to the contrast (Syringe A) and the extension to the saline (Syringe B). If the T-connector is connected to Syringe B, priming will not completely fill the connector tubing.
5. Connect the Prime Tube to the patient end of the tubing set, then prime the tubing by pressing Prime.
6. Ensure all air is purged.
7. Rotate the injector head downward.
8. Remove the Prime Tube, connect to the patient and press Check For Air.

Transfer Set Installation

- 1. Connect the transfer set by inserting the media container into the holder, remove the spike cover and insert the spike.
2. Remove the protective caps from the syringe and

- female luer of the transfer set.
3. Install the transfer set onto the end of the syringe, 1/4 turn to 1/2 turn maximum. Do not install with excessive force.
4. Fill the syringe by retracting the syringe piston/plunger. When complete, expel air from the syringe and fluid path.
NOTE: The transfer set valve may make a squealing sound during operation. This is considered normal.

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UNITED STATES

Bayer Medical Care Inc.
1 Bayer Drive
Indianola, PA 15051-0780
USA
Phone: +1 (412) 767-2400
+1 (800) 633-7231
Fax: +1 (412) 767-4120

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©2011-2014, 2017 Bayer. All Rights Reserved. Reproduction of this manual is strictly prohibited without express written consent of Bayer Medical Care Inc.

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Bayer, the Bayer Cross, MEDRAD, EnVision, Vistron, Qwik-Fit Syringe and FluiDots may be registered trademarks of Bayer in the US and other countries.

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For devices labeled for single use, please note: This product is intended for single use only. Do not resterilize, reprocess or reuse. The disposable devices have been designed and validated for single use only.

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Cautions

Component damage or leaks may occur if not installed properly. Ensure all connections are secure; do not overtighten.

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Syringe Installation:

- 1. Ensure that the piston(s) on the injector head are fully retracted.
2. Remove the new syringe(s) from the package and insert into the injector head Syringe Interface ensuring that the Keyed

Wings at the syringe base and openings in the Syringe Interface are aligned.

Note that the syringe is keyed to properly fit one way.

- 3. Twist the syringe 1/4 turn clockwise until the syringe clicks, locks, and the alignment marks on the syringe and injector head are aligned.

Syringe Filling:

NOTE: One person should fill and arm the injector. If you must change the injector operator, be certain that the second operator verifies that the syringe is properly filled and that any air is eliminated.

- 1. Tilt the injector head (or the syringe interface module) up.
2. Advance the plunger to full forward position.
3. Remove the protective covers from the syringe luer tip and attach a sterile filling device onto the tip of the syringe.

- a. Quick Fill Tube - raise the contrast bottle until the Quick Fill Tube is fully inserted.
b. Filling Device (spike) - prepare the bottle or bag of contrast and/or flushing solution.

NOTE: Use a Quick Fill Tube or equivalent device to reduce the volume and size of air bubbles drawn into the syringe during filling.

- 5. Aspirate contrast media into the syringe.
6. If the syringe has FluidDots indicators use them to help you detect the presence of contrast media in the syringe.

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- 1. Remove the filling device and expel air from the syringe.
2. Verify that the tubing is not kinked or obstructed.
3. (If your injector does not have a manual knob, proceed to Step 4.) Use the manual knob to advance the syringe plunger to provide a slow flow of contrast media through the connector tubing.

An absence of flow is an obvious indication of air in the fluid path.

- 4. Prime the connector tubing and make certain that the air is expelled.
5. Verify that all the air has been expelled from the syringe and fluid path.
6. Attach the connector tube to the vascular entry device.
7. Tilt the injector head (or syringe interface module) down.

Reinstalling a Syringe

If you remove a syringe from the injector and then wish to reinstall it, perform the following steps:

- 1. Insert the end of the syringe in the horizontal cutouts in the injector head.
2. Advance the piston until it is past the plunger feet and the piston and plunger interlock.
3. Rotate the syringe 1/4 turn clockwise until the syringe locks and alignment marks are positioned.
4. Proceed as normal by aspirating and dislodging any air bubbles.

Removing a Syringe

To remove the syringe when the procedure has been completed, rotate the syringe 1/4 turn counterclockwise and remove it from the injector head. Discard all used components.

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ENGLISH

Introduction: Read the information contained in this section. Understanding the information will assist you in operating the device in a safe manner.
Important Safety Notice: This device is intended to be used by individuals with adequate training and experience in diagnostic imaging studies.
Intended Use: The contents of this package are intended to be used in the delivery of contrast media or saline. They are indicated for single-use on one patient only with MEDRAD® Stellant Injectors.
Contraindications: These devices are not intended for multiple patient use, drug infusion, chemotherapy, or any other use for which the device is not indicated.
Restricted Sale: Rx Only

Warnings

Air embolization can cause death or serious injury to the patient. Do not connect a patient to the injector until all trapped air has been cleared from the syringe and fluid path. Carefully read the instructions for loading and the use of FluIDots indicators (where applicable) to reduce the chance of air embolism.

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For devices labeled for single use, please note: This product is intended for single use only. Do not resterilize, reprocess or reuse. The disposable devices have been designed and validated for single use only. Re-use of the single use disposable devices pose risks of device failure and risks to the patient. Potential device failure includes significant component deterioration with extended use, component malfunction, and system failure. Potential risks to the patient include injury due to device malfunction or infection as the device has not been validated to be cleaned or re-sterilized.

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Automatic prime will not function properly and an air injection may result if SJS-LP-60-T-J is used without an adapter. Do not use SJS-LP-60-T-J if an adapter is not installed on the injector.

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Installing a Syringe
Insert the syringe until it snaps into place. The piston automatically engages the plunger, and advances it to the front of the syringe (most models). To reduce the size and quantity of air bubbles drawn into the syringe during loading, a Fluid Dispensing Device from Bayer, is recommended.
Operator vigilance and care, coupled with a set procedure, is essential to minimizing the possibility of an air embolism. Point the injector head up during loading. Point the injector head down during an injection.
To help avoid an air injection, syringes from Bayer are equipped with FluIDots indicators. FluIDots indicators should be observed as part of an arming procedure. When the FluIDots are viewed through an empty

syringe, the dots appear as small narrow ellipses. When viewed through a full syringe, the dots become larger, almost round. See below:

Empty Syringe **Filled Syringe**
To minimize air embolization risks, ensure that one operator is designated the responsibility of filling the syringe(s). Do not change operators during the procedure. If an operator change must occur, ensure that the new operator verifies that the fluid path is purged of air.

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MEDRAD Stellant Manual Loading:

1. Install a new syringe.
2. Install the spike or QFT onto the end of the syringe. 1/4 turn to 1/2 turn maximum. Do not install with excessive force.
3. Insert the spike or QFT into the fluid source and fill the syringe with fluid, using the load buttons.
4. Expel air.
5. Remove spike or QFT. Connect the disposable tubing set.
6. Follow the instructions in the section "Connector Tube Installation."

MEDRAD Stellant Integral Autoload with a spike or QFT:

1. Set and lock protocol.
2. Install a new syringe.
3. Install the spike or QFT onto the end of the syringe. 1/4 turn to 1/2 turn maximum. Do not install with excessive force.
4. Insert the spike or QFT into the fluid source and press Autoload.
5. Press Fill.
6. Expel air.
7. Remove spike or QFT. Connect the disposable tubing set.
8. Follow the instructions in the section "Connector Tube Installation."
9. Press Start/Hold.

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Removing a Syringe(s)

1. Disconnect the disposable tubing set from the vascular entry device. The disposable tubing set does not need to be disconnected from the syringe.
2. Rotate the syringe approximately 1/4 turn counter-clockwise and gently pull the syringe out of the injector head, discarding the syringe with disposable tubing set.

Note: Once the syringe is removed from the injector, the piston will automatically retract (most models).

Note: In order to remove the syringe, the last piston motion must be in the forward direction, which is typical. If you cannot remove the syringe, rotate the injector head manual knob approximately one revolution in the forward direction, then repeat Step 2.

Connector Tube Installation

1. Remove the connector tube from the package, removing the dust covers on the luer fittings.
2. Ensure all air is purged from the syringe.
3. Attach the connector tube to the syringe, 1/4 turn to 1/2 turn maximum. Do not install with excessive force.
4. Ensure that the connector luer fitting is secured to the tip of the syringe and verify that the tubing is not kinked or obstructed.

5. If you are using a connector tube with T-connector, attach the straight portion of the T-connector to the contrast (Syringe A) and the extension to the saline (Syringe B). If the T-connector is connected to Syringe B, priming will not completely fill the connector tubing.
6. Connect the Prime Tube to the patient end of the tubing set, then prime the tubing by pressing Prime.
7. Ensure all air is purged.
8. Rotate the injector head downward.
9. Remove the Prime Tube, connect to the patient and press Check For Air.

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Biological contamination can result from reusing disposable items or failure to follow aseptic technique.

Syringe sterility will be compromised, and patient infection may result, if the plunger is removed from the syringe.

Bacterial contamination can occur if syringes are used to store contrast media. Use loaded syringes immediately.

For devices labeled for single use, please note: This product is intended for single use only. Do not resterilize, reprocess or reuse.

Do not use if sterile package is opened or damaged. Patient or operator injury may result if package is opened or damaged.

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Patient or Operator injury may result from contrast media leaks or tubing ruptures. Ensure that the fluid path is open; do not exceed pressures identified on the front of the package.

Operator injury could result from using a sharp spike. Use care in handling and inserting the spike into the bottle.

Serious injury may result from syringe failure. Do not retract pistons with connector tubing installed.

Cautions

Component damage or leaks may occur if not installed properly. Ensure all connections are secure, do not overtighten.

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Syringe Installation:

- 1. Ensure that the piston(s) on the injector head are fully retracted.
2. Remove the new syringe(s) from the package and insert into the injector head...

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Syringe Filling:

NOTE: One person should fill and arm the injector. If you must change the injector operator, be certain that the second operator verifies that the syringe is properly filled...

- 1. Tilt the injector head up.
2. Advance the plunger to full forward position.
3. Remove the protective cover(s) from the syringe luer tip...

- a. Spike(s) - The SQK 65S, SQK 65VS, and SSQK 65/115VS kits contain two spikes. Use the smaller spike to load contrast media syringe A and the larger spike to load saline syringe B.
b. FFA - The FFA is included in the SQK 65VA kit.

NOTE: Use a filling device to reduce the volume and size of air bubbles drawn into the syringe during filling.

- 5. Aspirate contrast media into the syringe.
6. The syringe has FluidDots indicators to help you detect the presence of contrast media in the syringe.

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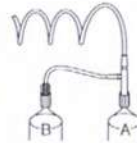
- 1. Remove the filling device and expel air from the syringe.

Do not hit the syringe to remove air bubbles. Reverse the plunger 3 to 5 ml then expel the remaining air.

- 2. The MEDRAD Spectris MR Injector uses two syringes and low pressure connector tubing with a 'T' connector. Attach the straight portion to the contrast media syringe A and the extension to the saline syringe B.

- 3. Verify that the tubing is not kinked or obstructed.
4. Use the manual knob to prime the connector tubing and make certain that the air is expelled.

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- 5. Verify that all the air has been expelled from the syringe and fluid path.
6. Attach the connector tube to the vascular entry device.
7. Tilt the injector head down.

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Removing a Syringe

To detach the syringe when the procedure has been completed, rotate the syringe 1/4 turn counterclockwise and remove it from the injector head.

NOTE: Do not use unless the plunger feet are properly engaged with the piston.

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ENGLISH

Introduction: Read the information contained in this section. Understanding the information will assist you in operating the device in a safe manner.

Important Safety Notice: This device is intended to be used by individuals with adequate training and experience in diagnostic imaging studies.

Indications for use: The contents of this package are intended to be used in the delivery of contrast media. These devices are indicated for single-use only with injectors from Bayer.

Contraindications: These devices are not intended for multiple-use, drug infusion, chemotherapy, or any other use for which the device is not indicated.

Restricted Sale: Federal (USA) law restricts these devices for sale by or on the order of a physician.

Warnings

For devices labeled for single use, please note: This product is intended for single use only. Do not resterilize, reprocess or reuse. The disposable devices have been designed and validated for single use only. Re-use of the single use disposable devices pose risks of device failure and risks to the patient.

Air embolization can cause death or serious injury to the patient. Do not connect a patient to the injector until all trapped air has been cleared from the syringe and fluid path.

Biological contamination can result from reusing disposable items or failure to follow aseptic technique. Properly discard disposable items after use.

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Bacterial contamination can occur if syringes are used to store contrast media. Use loaded syringes immediately. Do not store loaded syringes for later use. Discard unused loaded syringes.

Do not use if sterile package is opened or damaged. Patient or operator injury may result if package is opened or damaged, or if damaged components are used.

Patient or Operator injury may result from contrast media leaks or tubing ruptures. Ensure that the fluid path is open, do not exceed pressures identified on the front of the package.

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- 1. Ensure that the piston on the head is fully retracted. To retract the piston(s), press the Reverse Load Switch on the injector head.
2. Set the Mechanical (safety) Stop to zero in the full forward position (applies to MEDRAD Mark IV and MEDRAD CT-100 Series Injectors only). Position the empty Pressure Jacket in the top of the turret. If there is a syringe in the top pressure jacket, remove and discard.
3. Remove the syringe from its package and

slide it in the top Pressure Jacket (the flat side of the flange must face upward). This positions the nut with its cutout facing up for easier hub attachment.

- 4. Fully rotate the turret to place the syringe plunger in front of the piston.

Note: The turret must be fully rotated to the closed position to ensure syringe/syringe plunger alignment to prevent contrast leakage.

The following are general guidelines only. For additional instructions, refer to the Operation Manual for your Liebel Flarsheim injector.

Liebel Flarsheim 6000 and 6000 CT Injectors:

- 1. Ensure that the piston on the head is fully retracted.
2. Unlock pressure jacket and rotate downward.
3. Remove syringe from its package and insert syringe into pressure jacket, making sure to position alignment key with pressure jacket slot.
4. Rotate pressure jacket plate upward and latch by turning knob fully clockwise.
5. Advance piston to engage syringe plunger. Liebel Flarsheim Angiomat Injector:

- 1. Press the LOAD button on the console. Press the LOAD button on the head to fully retract the plunger.
2. Set the mechanical stop at zero.
3. Open the turret door so the pressure jacket is ready to accept a syringe. If there is a syringe in the pressure jacket, remove and discard.
4. Remove syringe from its package and insert syringe into pressure jacket.
5. Close and secure the turret.
6. Press the UNLOAD button to advance the piston forward.

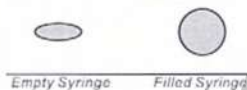
Syringe Filling:

Note: One person should fill and arm the injector. If you must change the injector operator, be certain that the second operator verifies that the syringe is properly filled and that any air is eliminated.

- 1. Tilt the injector head up.
2. Press and hold the forward motion control until the piston is fully advanced.
3. Remove the protective covers from the syringe luer tip and attach a sterile filling device onto the tip of the syringe (Quick Fill Tube or other filling device).
4. Transfer the contrast media into the syringe using a Quick Fill Tube. Raise the contrast bottle until the Quick Fill Tube is fully inserted. The end of the Quick Fill Tube should be near the bottom of the contrast bottle.

Note: Use a Quick Fill Tube or equivalent device to reduce the volume and size of air bubbles drawn into the syringe during filling. It is more difficult to remove the air bubbles if you use smaller diameter tubes, such as a catheter-over-needle, or a tube longer than 10 in. (25cm).

- 5. Aspirate contrast media into the syringe.
6. Press and hold the REVERSE piston motion key until the syringe is filled with the volume of contrast media desired.
7. Use the FluidDots indicators to help you detect the presence of contrast media in the syringe. Verify that the FluidDots indicators are round in the filled portion of the syringe. The rounded shape of the FluidDots indicators varies according to the type of contrast media, but an oblong shape indicates the presence of air. Rounded FluidDots indicators do not indicate the total absence of air bubbles in the syringe tip

Expelling Air and Attaching the Connector Tube:

After you fill the syringe, you must expel all air from the syringe and then attach the connector tubing. To do so:

- 1. Remove the filling device and expel air from the syringe by manually advancing the piston so that the contrast media is in the tip of the syringe.

Gently tap the base of the pressure jacket to dislodge the air bubbles. Expel remaining air.

- 2. Attach the connector tube to the syringe:

- a. FasTurn - Place the connector tube hub in the syringe nut's cutout. Turn the syringe nut clockwise approximately 1 1/4 turns until secure while holding the hub stationary.
b. Push-N-Turn - Place the connector tube hub in the syringe nut's cutout. Push the syringe nut towards the injector head until it snaps; turn the nut clockwise approximately 1 turn until secure.

NOTE: Position the hubs so that when the syringe nut is secure, the wings are across the cutout, not with one wing in the cutout.

- 3. Verify that the tubing is not kinked or obstructed.

- 4. Use the manual knob to advance the syringe plunger to provide a slow flow of contrast media through the connector tubing.

An absence of flow is an obvious indication of air in the fluid path

- 5. Prime the connector tubing and make certain that the air is expelled.
6. Verify that all of the air has been expelled from the syringe and fluid path.
7. Attach the connector tube to the vascular entry device.
8. Tilt the injector head down.

Removing a Syringe:

To remove the syringe when the procedure has been completed, fully retract the piston and rotate the turret counter-clockwise. Remove the syringe from the pressure jacket. Discard all used components.

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The State of Maryland

Office of the Secretary of State

This Apostille is not valid for use anywhere within the United States of America, its territories or possessions.
This Apostille does not certify the content of the document for which it is issued.

Apostille

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1. Country: United States of America
This public document
2. has been signed by **Barbara H. Meiklejohn**
3. acting in the capacity of **Clerk of the Circuit Court for Montgomery County**
4. bears the seal/stamp of the **Circuit Court for Montgomery County**

Certified

5. at Annapolis, Maryland
6. the **20th day of July, 2020**
7. by The Secretary of State of Maryland

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I, Barbara H. Meiklejohn, Clerk of the Circuit Court for Montgomery County, Maryland, a court of record, hereby certify that VERA V CHAWLA was a commissioned/ appointed and qualified Notary Public commencing on the 9th day of August, 2016.

In Testimony Whereof, I have hereunto set my hand and affixed the seal of the Circuit Court for Montgomery County this 13th day of July, 2020.

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2141 P Street NW, Suite 103, Washington, DC 20037

Affidavit

State of Maryland)
) SS:
County of Montgomery)

Before me, the undersigned notary public, this day, personally, appeared to me known or proven, who being duly sworn according to law, deposes the following:

I, Yoshiko Yano, hereby certify that the attached document is

- an original document
- a true copy of an original document

to the best of my knowledge and belief.

[Signature]
Signed

Subscribed and sworn to before me this

26 day of June, 2020

VEFA V. CHAWLA
NOTARY PUBLIC
MONTGOMERY COUNTY
MARYLAND
MY COMMISSION EXPIRES JULY 29, 2020

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Best regards,

Emily Bartolomeo

Emily Bartolomeo
Manager Global Regulatory Strategy Devices
Bayer U.S. LLC
Pharmaceuticals

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1. Medical Device General Information

MEDRAD® syringes for X-ray contrast agents and saline solution for Mark V ProVis injection systems:

150-FT-Q composed of:

- Disposable sterile syringe 150 ml;
- Quick fill tube.

200-FT-Q composed of:

- FasTurn disposable sterile syringe 200 ml;
- Quick fill tube

2. Medical device purpose

The kits are intended for administration of contrast agents or saline solution.

3. Operation conditions

Product	Conditions
150-FT-Q	Temperature: 15°C to 30°C
	Humidity: 5% to 100%
	Atmospheric pressure: 70 kPa to 110 kPa
200-FT-Q	Temperature: 15°C to 30°C
	Humidity: 20% to 90%
	Atmospheric pressure: 70 kPa to 110 kPa

4. Medical device technical description

150-FT-Q, 200 FT-Q		
Parameter	Value	
	Sterile disposable syringe 150 ml	FasTurn sterile disposable syringe 200 ml
		
Syringe filling volume, ml	0-150	0-200
Syringe barrel length, mm	231.39	234.44
Syringe barrel inner diameter, mm	40.49	46.74
Largest outer diameter of the syringe barrel, mm	52.32	59.44
Barrel cone diameter, mm		
- maximum	7.95	7.95
- minimum	2.67	2.67
Quick fill tube.		

Outer diameter, mm	5.49
Inner diameter, mm	4.17
Length, mm	190.50

5. Transportation and storage conditions

Product	Conditions
200-FT-Q/ 150-FT-Q	Temperature: -40°C to 70°C
	Humidity: 5% to 95% non-condensing
	Atmospheric pressure: 70 kPa to 110 kPa

6. Shelf life

Device is supplied sterile. Sterilized by radiation.
Shelf-life (sterilization validity):

150-FT-Q, 200-FT-Q: 3 years from the date of production/sterilization

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8. Warranty liabilities

The warranty lifetime is not provided for due to the specifics of the medical device. Manufacturer warrants that the products are free from defects for the entire device shelf-life.

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Certified

5. at Annapolis, Maryland
6. the **20th day of July, 2020**
7. by The Secretary of State of Maryland
8. No. 520743

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www.radiology.bayer.com

Best regards,

Kate Ueda
Senior Manager Regulatory Strategy, Device LCE
Bayer U.S. LLC
Pharmaceuticals

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1. Medical device designation

MEDRAD® syringes for X-ray contrast agents and saline solution for Vistron CT injection systems:

CTP-200-FLS Disposable kit for CT injection system, composed of:

- Syringe Qwik-Fit 200 ml;
- Coiled low pressure connector tube;
- Quick fill tube

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3. Possible undesirable effects of the medical device use

No possible undesirable effects identified by manufacturer. All possible warnings that are necessary for the safe use of the product are indicated in the instructions for use.

4. Operation conditions

	Conditions
Temperature	10°C to 40°C
Humidity	5% to 100%
Ambient pressure	from 48 kPa to 110 kPa

5. Medical device technical description

Parameter	Value	
	Syringe Qwik-Fit 200 ml	
		
Syringe filling volume, ml	0-200	
Syringe barrel length, mm	184.73	
Syringe barrel inner diameter, mm	46.84	
Largest outer diameter of the syringe barrel, mm	64.01	
Low pressure twisted connecting tube		
		
Outer diameter, mm	2.54	
Inner diameter, mm	1.52	

Wall thickness, mm	0.51
Length, mm	1574.8
Quick fill tube	
	
Outer diameter, mm	5.49
Inner diameter, mm	4.17
Length, mm	190.50

6. Transportation and storage conditions

	Conditions
Temperature	-40°C to 70°C
Humidity	5% to 100% non-condensing
Ambient pressure	from 48 kPa to 110 kPa

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Shelf-life (sterilization validity):

CTP-200-FLS: 5 years from the date of production/sterilization

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The State of Maryland

Office of the Secretary of State

This Apostille is not valid for use anywhere within the United States of America, its territories or possessions.

This Apostille does not certify the content of the document for which it is issued.

Apostille

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- 1. Country: United States of America
This public document
- 2. has been signed by **Barbara H. Meiklejohn**
- 3. acting in the capacity of **Clerk of the Circuit Court for Montgomery County**
- 4. bears the seal/stamp of the **Circuit Court for Montgomery County**

Certified

- 5. at Annapolis, Maryland
- 6. the **20th day of July, 2020**
- 7. by The Secretary of State of Maryland
- 8. No. 520739

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State of Maryland, Montgomery County, Sct.

In the Office of the Clerk of the Circuit Court for Montgomery County

I, Barbara H. Meiklejohn, Clerk of the Circuit Court for Montgomery County, Maryland, a court of record, hereby certify that **VERA V CHAWLA** was a commissioned/ appointed and qualified Notary Public commencing on the 9th day of August, 2016.

In Testimony Whereof, I have hereunto set my hand and affixed the seal of the Circuit Court for Montgomery County this 16th day of July, 2020.

A handwritten signature in black ink, appearing to read "Barbara H. Meiklejohn", written over a horizontal line.

Barbara H. Meiklejohn

Clerk of the Circuit Court for Montgomery County

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July 13, 2020

Pharmaceuticals
Global Regulatory Affairs

1 Bayer Drive
Indianola (PA), United States
15051

www.bayer.com

To whom it may concern,

To be submitted to the Competent Health Authorities of the Russian Federation.

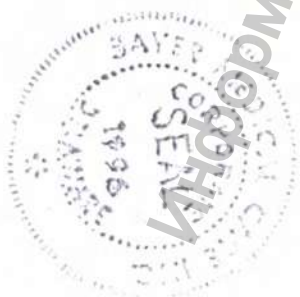
We, Bayer Medical Care Inc., USA (1 Bayer Drive Indianola, Pennsylvania 15051-0780, USA) hereby state that attached document (addendum to the instruction for use) about MD "Шприцы MEDRAD® для рентгеноконтрастных / магнитоконтрастных веществ и физиологического раствора к системам инъекционным MEDRAD® Stellant Sx, MEDRAD® Stellant D, MEDRAD® Stellant DualFlow, MEDRAD® Stellant D с рабочей станцией Certegra, MEDRAD® Spectris Solaris EP, Mark V ProVis, Vistron CT, в наборах" is true.

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Addendum to the instructions for use

1. Medical device designation

MEDRAD® syringes for magneto-contrast agents and saline solution for the MEDRAD® Spectris Solaris EP system:

SSQK 65/115VS Disposable kit for injection system 65/115 MR for MRI, composed of:

- Qwik-Fit Syringe 65 ml for contrast media;
- Qwik-Fit Syringe 115 ml for saline;
- Small spike for contrast;
- Large spike for saline;
- Low pressure connector tube 243.84 cm for 65/115 MR injector system

2. Operation conditions

	Conditions
Temperature	10°C to 40°C
Humidity	20% to 90%
Ambient pressure	from 48 kPa to 110 kPa

3. Transportation and storage conditions

Product	Conditions
Temperature	-25°C to 70°C
Humidity	5% to 100% non-condensing
Ambient pressure	5% to 100% non-condensing

4. Technical characteristics

Parameter	Value	
	Qwik-Fit Syringe 115 ml for saline solution;	Qwik-Fit Syringe 65 ml for contrast agent
		

Syringe filling volume, ml	0-115	0-65
Syringe barrel length, mm	167.06	165.53
Largest outer diameter of the syringe barrel, mm	54.1	45.97
Syringe barrel inner diameter, mm	35.43	26.29
Plunger cap width, mm	17.12	13.72
Needle length without cap, mm	61.98	35.81
Scale division, mm	1	1
Capacity between graduation lines with numbers, ml	10	5
Low pressure connecting tube 243.84 cm for injection system 65/115 MR		
Straight tube length, mm	190.5	
Straight tube outer diameter, mm	2.54	
Straight tube inner diameter, mm	1.52	
Straight tube wall thickness, mm	0.51	
Twisted tube length, mm	2438.4	
Twisted tube outer diameter, mm	3.18	
Twisted tube inner diameter, mm	1.91	
Twisted tube wall thickness, mm	0.64	

5. Shelf life

Device is supplied sterile. Sterilized by radiation.

Shelf life (sterilization validity):

SSOK-65/115 VS: 5 years from the date of production/sterilization

6. Disposal Procedure

Used syringes must be disposed of in a biohazard waste container in accordance with common medical practice and local regulations.

Page 4 of 7

If package is damaged the device is disposed in accordance with the A class medical waste treatment rules.

Used device is disposed in accordance with the B class medical waste treatment rules.

7. Warranty liabilities

The warranty lifetime is not provided for due to the specifics of the medical device.

Manufacturer warrants that the products are free from defects for the entire device shelf-life.

Manufacturer bear no responsibility nor liability in case of misuse or non-adherence to the Instructions For Use.

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Certified

- 5. at Annapolis, Maryland
- 6. the **20th day of July, 2020**
- 7. by The Secretary of State of Maryland

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A handwritten signature in black ink, appearing to read "Barbara H. Meiklejohn", is written over a horizontal line.

Barbara H. Meiklejohn
Clerk of the Circuit Court for Montgomery County

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[Signature]
Signed

Subscribed and sworn to before me this
14 day of July, 2020.

VEENA V. CHAWLA
NOTARY PUBLIC
MONTGOMERY COUNTY
MARYLAND
MY COMMISSION EXPIRES JULY 23, 2020

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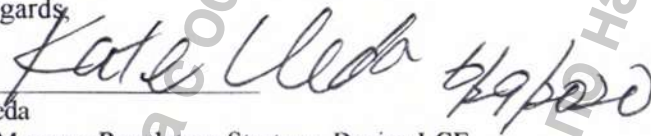
1. Medical device designation

MEDRAD® syringes for X-ray contrast agents and saline solution for injection systems MEDRAD® Stellant Sx, MEDRAD® Stellant D, MEDRAD® Stellant DualFlow, MEDRAD® Stellant D with Certegra workstation, in kits:

SDS-CTP-QFT Disposable sterile syringe in the kit:

- Syringes 200 ml - 2pcs;
- T connector tube 152.4 cm;
- Prime tube;
- Quick fill tube.

SDS-CTP-SPK Disposable sterile syringe in the kit:

- Syringes 200 ml - 2pcs;
- T- connector tube 152.4 cm;
- Prime tube;
- Spikes - 2 pcs.

SSS-CTP-QFT Disposable sterile syringe in the kit:

- Syringe 200 ml;
- Low pressure connector tube;
- Prime tube;
- Quick fill tube.

SSS-CTP-SPK Disposable sterile syringe in the kit:

- Syringe 200 ml;
- Low pressure connector tube;
- Prime tube;
- Spike

SSS-QFT Disposable sterile syringe in the kit:

- Syringe 200 ml;
- Quick fill tube.

SSS-SPK Disposable sterile syringe in the kit:

- Syringe 200 ml;
- Spike

2. Medical Device Purpose/Indications

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1. Medical device designation

MEDRAD® syringes for X-ray contrast agents and saline solution for injection systems MEDRAD® Stellant Sx, MEDRAD® Stellant D, MEDRAD® Stellant DualFlow, MEDRAD® Stellant D with Certegra workstation:

SDS-TRF-CV Disposable sterile syringe in the kit:

- 1 Syringes 200 ml - 2pcs;
- 2 T-connector tube 152.4 cm;
- 3 Prime tube;
- 4 Check valve transfer sets - 2 pcs.

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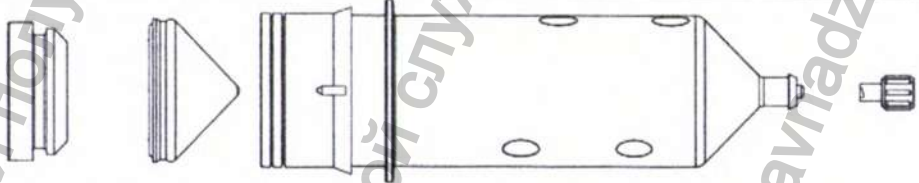
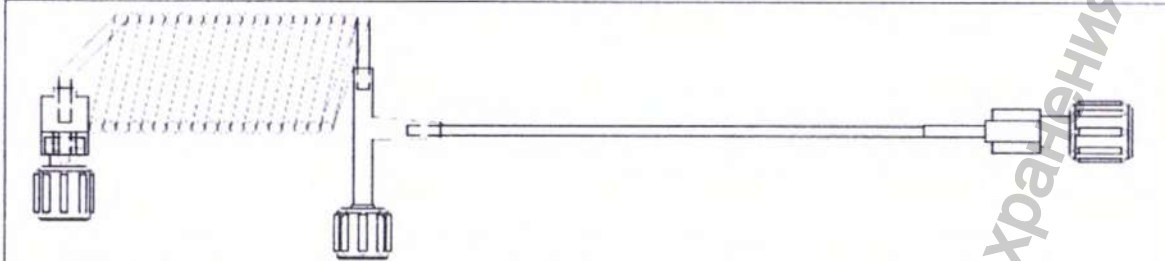
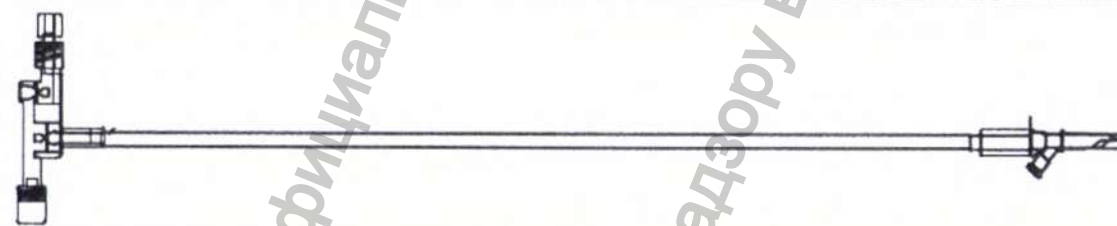
Twisted tube	Length, mm	1524
	Outer diameter, mm	2.54
	Inner diameter, mm	1.52
Straight tube	Length, mm	152.4
	Outer diameter, mm	2.54
	Inner diameter, mm	1.52
Tube for pumping air;		
		
Tube	Length, mm	111.51
	Outer diameter, mm	5.49
	Inner diameter, mm	4.17
Filter	Filter length, mm	12.32
Adapters with a check valve		
		
Needle	Needle body length, mm	61.21
	Needle tip length, mm	33.65
	Needle diameter, mm	6.6
Tube	Length, mm	1143
	Outer diameter, mm	4.06
	Inner diameter, mm	2.79

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	Conditions
Temperature	-25°C to 70°C
Humidity	5% to 100% non-condensing
Ambient pressure	from 48 kPa to 110 kPa

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SDS-TRF-CV: 4 years from the date of production/sterilization

7. Warranty liabilities

The warranty lifetime is not provided for due to the specifics of the medical device.
Manufacturer warrants that the products are free from defects for the entire device shelf-life.

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Bayer, the Bayer Cross, MEDRAD, Spectris Solaris, MEDRAD Spectris Solaris, Qwik-Fit Syringe and FluiDots may be registered trademarks of Bayer in the US and other countries.
© 2011-2014, 2017 Bayer. All Rights Reserved. Reproduction of this manual is strictly prohibited without express written consent of Bayer Medical Care Inc. / © 2011-2014, 2017 Bayer. Tous droits réservés. La reproduction de ce manuel est strictement interdite sans le consentement écrit explicite de Bayer Medical Care Inc.